I

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GB

Ask for information to the surgeon on how to behave immediately after surgery and in the following days, such as the type and the frequency of cold compresses to be carried out.

No physical exertion and/or sauna for the first 24 hours.

No alcohol, no nicotine, no tea or coffee in the first days.

No chewing in the area where the dental implant is placed.

Patient Home Care & Maintenance Instructions

1. Regularly brush the implants to keep them free of plaque and debris. Treat them very much like teeth.
2. Maintain an appropriate and continuous oral hygiene.
3. Clean and sanitize dentures regularly. We recommend brushing and soaking overnight using a solution of equal parts vinegar and water. Avoid common cleaning chemicals including bleach, hydrogen peroxide, and solvents of any kind.
4. Return for regular check-ups and professional cleaning.
5. Contact your dentist if you experience sore spots, notice loss of retention, dentures rock back and forth, or an O-ring is lost or damaged.
6. Advise medical professionals about the presence dental implants prior to MRI imaging. In most cases, titanium dental implants will not interfere with MRIs.

Zatražite informacije od kirurga o tome kako se ponašati neposredno nakon zahvata te u narednim danima, kao što su vrsta i učestalost hladnih kompresa koje treba upotrebljavati.

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